



aks support strap

The aks support strap incorporates various auxiliary functions. It can be used as an aid for transferring to a seat. In doing so, the complete sequence of movements is controlled by the caregiver. Assistance with standing up and sitting down can also be supported by the caregiver without back strain.

The aks support strap is also a valuable aid for walking with unsteady patients without having to hold their arm (prevention of pain).

Multiple grip loops make supporting and easier use by the caregiver possible. An adjustable and automatically centring fastener on the front side ensures a secure fit. Four sizes enable use with many different abdominal girths.

Technical Data

Outer material: polyester Inner material: non-slip PU-coated fabric, washable at 60 °C; do not dry in a dryer Fastener: push-fit fastening, self-centring

Order numbers

Size S, waist size 60-80 cm – 62001 Size M, waist size 75-95 cm – 62002 Size L, waist size 90-110 cm – 62003 Size XL, waist size 110-130 cm – 62004

aks lifting sling

The aks lifting sling is an aid whose intended purpose is easy on the back support of the patient when uprighting from and setting down on a seat. This effectively makes the daily work of care personnel easier.

Two sizes with two different length grip loop pairs each are available and can be used for different size patients.

Technical Data

Outer material: polyester Inner material: non-slip PU-coated fabric, washable at 60 °C; do not dry in a dryer

Order numbers

Size M, 60×20 cm (without grip loops) -62005 Size L, 90×25 cm (without grip loops) -62006









aks pelvic belt comfort

The aks pelvic belt comfort provides optimum support of the lap positioning for partially mobile individuals in the sitting position. It is easy and quick to handle and supports partially mobile people in wheelchairs or other seating. Patients are effectively protected against slipping forwards. Stability, workmanship quality and fit provide high wearing comfort without the material cutting into the thighs.



Technical Data

Material: 100% cotton, lockstitch jersey Inner-side coating: TEXTHANE, water-repellent and breathable Washable: up to 30 °C; do not dry in a dryer

Order numbers

Size S, waist size 60-80 cm – 69803/1 Size M, waist size 75-95 cm – 69803/2 Size L, waist size 90-110 cm – 69803/3







aks sliding and turning sheet

The aks sliding and turning sheets are made of silicone-coated fabric that effectively reduces the friction resistance during the repositioning of patients. This makes the corresponding activities of care personnel easier. Different sizes of the aks sliding and turning sheets can be used depending on the restriction of the patient's mobility. The body regions to be supported are decisive here.


The large number of possible uses covers repositioning of normal, heavy and very heavy immobile patients from abdomen, back or side position to any required position. Good patient care simply becomes more feasible.

Technical Data

Outer material: silicone-coated fabric, green washable at 60° C; do not dry in a dryer

Order numbers

Size 195 x 70 cm – 62011 Size 140 x 110 cm – 62010 Size 130 x 70 cm – 62009 Size 80 x 70 cm – 62008 Size 48 x 48 cm – 62007

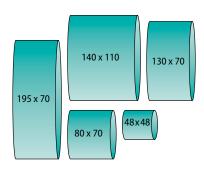

aks sliding and turning sheet with grip loops

This design combines all positive product characteristics of the aks sliding and turning sheets and may potentially be used for particularly heavy and very heavy patients. The grip loops here provide as low friction resistance as possible while in use and thus make movement of these patients possible. With more and more people suffering from adiposity, the use of such sheets is becoming increasingly more important.

Technical Data

Outer material: silicone-coated fabric, green Grip loops: Polyester sling, red washable at 60 °C; do not dry in a dryer

Order numbers

Size 195 x 70 cm – 62014 Size 140 x 110 cm – 62013 Size 150 x 70 cm – 62012

